

IMPORTANT:

Please read before
opening & unpacking the
AquaDoula tub.

Follow the enclosed
instructions for safe use,
cleaning & proper assembly.

Before unpacking, please note how the equipment is packed in its box, so repacking the tub when you are finished will be easier.

SEE REVERSE SIDE

Set-Up of AquaDoula Tub

Thank you for choosing to rent an AquaDoula Spa!

The following are recommendations intended to facilitate ease of use of the AquaDoula tub:

Before unpacking, please note how it is packed in its box to ensure that re-packing the tub when you are finished will be easier. Unpack the equipment mindfully, exercising care to avoid damaging the materials and contents.

Please read and follow the enclosed Instruction Guide prior to and during assembly.

Practice assembling the tub at least once upon receiving to familiarize yourself with its components. You do not need to fill with water as a test run, **but make sure the hose fits the faucet with the particular adapter provided. If it does not fit or work properly, you must purchase an adapter that does or find another faucet to use!**

Filling the tub and maintaining an accurate temperature read:

Always FILL the tub from a **hot water tap** with the clean white lead-free hose provided or your own new hose. Do not use the reusable white hose for draining the tub.

Do not plug in the heating element before the tub is full of water.

Important: You must fill the tub with warm water at the desired temperature, roughly 99° F. The heating system is designed only to *maintain* the water temperature. Do not put water less than 95° F in the tub expecting the heater to heat the water. It is too much for the system and will not work.

To maintain an accurate temperature reading, you must ensure that the water in the tub has been thoroughly mixed, both when filling as well as when standing unused. Thermal layers occur when the water has been standing in the tub, especially when the Insul-Cover is left on. The thermometer floats at the top, reading just the first six inches or so. Hot water naturally rises to the top, while cold water naturally sinks to the bottom and this is where the temperature probe is located. Be sure to keep the water mixed and you will have more success at maintaining an accurate temperature. This means stirring the water every hour or so if it is left standing.

Once the tub has been used, DO NOT place the Insul-Cover on the tub.

After using the tub:

Begin the draining and cleaning process as outlined in the Instruction Guide in the section called "Disassembly." It is best to empty the tub immediately after use after removing any floating particles. **Use the dark hose for emptying the tub, and ONLY for emptying the tub;** or your personally purchased hose.

After the tub is drained and the disposable liner is disposed, follow the instruction as outlined on the document titled, "Break-Down/Cleaning"

Your attention to these considerations is greatly appreciated!!

If you have any questions, please call Esther at (510) 479-0905.